



## SET UP FUNCTIONS DSD-201

The following table lists the various set up functions that can be run on the DSD-201. The table shows the entry number and the corresponding function. Press SET-UP, the entry number; then ENTER.

<b>ENTRY NUMBER</b>	<b>FUNCTION</b>
1	Load disinfectant solution from basin
2	Set Date
3	Set Time
4	Display software version number
5	Input a custom program
6	Water line disinfection
7	Override disinfectant cycle warning
8	Display log information
9	Disable logging
10	Clear the log
11	Clear the disinfectant cycle count
13	Display temperatures
14	Set the heater ON time
15	Set the heater OFF time
16	Display the disinfectant cycle count
17	Display the cycle time remaining
18	Display the state time
21	Print the entire log
22	Cancel print run
25	Print the last disinfection cycle
28	Set delayed start time
29	Enable delayed start
33	Enable automatic log printout

[www.minntech.com/medivators](http://www.minntech.com/medivators) • Toll Free: 1-800-444-4729